

Quick Start Guide



smyle™ mouse

Head / Face Controlled Mouse

Software for Hands-free & Voice-free Computer Control via a Webcam

Includes:

1. Head Pointer
2. Smile & Dwell Clicker
3. Adaptive Switch Emulator

Award Winning & Patented



U.S. Patents: 9,013,264 & 9,785,242
Multiple Patents Pending

How to Setup

1. Download latest from:
<https://smylemouse.com/downloads/SmyleMouseInstaller>
2. Run the installer
3. Run **Smyle Mouse** to register your device


Register Your Device

First Name:*
Last Name:*
Email Address:*
Product Key Or Order Id: (Leave blank to use in Trial mode)
Register


4. Enter your **Product Key** if available

Getting Ready to Use

Placement

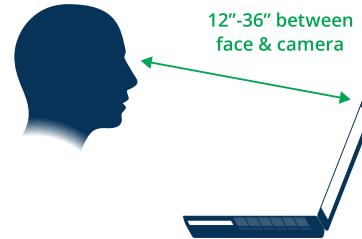

Orientation

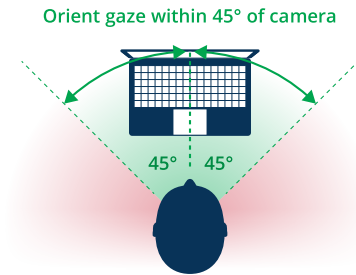

Both eyes must be visible to the webcam

Calibration

10-second process automatically starts at launch.

1. Hold head steady
2. Smile moderately (Not too much or too little)
3. Open mouth

Basics of Use

Instructions

Pointing: Turn your head to move the mouse.

Clicking: Smile briefly while holding head steady.

Dragging: Smile & turn your head while smiling.

User Interface

